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*Massa adesiva acrilica*

*Non sterile*

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## GB Dream® K | Elastic adhesive bandage for proprioceptive taping

### Instructions for use:

Before applying Dream® K, clean the skin with soap and water, then dry accurately. In case thick body hairs are present, shave the part before applying the tape. Body temperature helps to increase the adherence, so it is recommended to apply Dream® K at least 30-60 minutes before physical activity. To remove the tape, according to the skin structure, hold firmly the skin with one hand while carefully and steadily remove Dream® K with the other hand (avoid sudden movements).

### Warning:

Dream® K must be applied to the patient and used only by qualified healthcare professionals (i.e. Doctors, Physiotherapists, Sport Masseurs). Do not apply to red or irritated skin or on wounds. After applying the tape, if redness, irritation or itching of the skin occur, remove it immediately. Although the product is dermatologically tested, if the skin is known to be particularly delicate and sensitive, or if there is a predisposition for allergies to adhesives, certain types of fabric or colours, it is recommended to test a few centimetres of the product on the skin before application. Dream® K is affected by humidity.

Store in a cool dry place away from direct heat. Keep out of

the reach of children. Do not use if packaging is damaged.

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